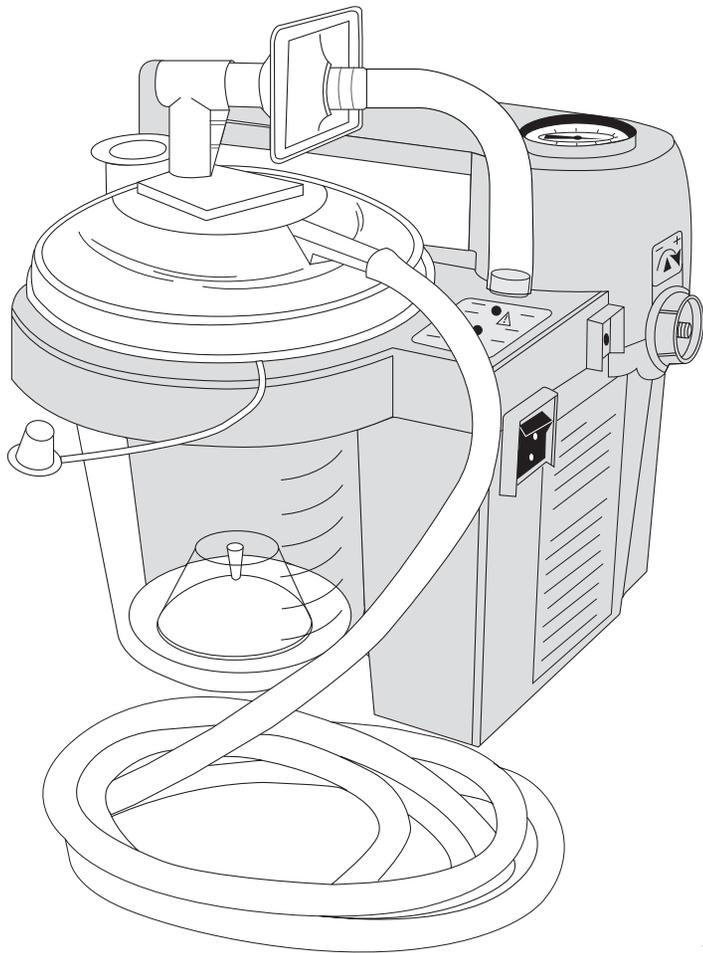
DEVILBISS

7305 Series
Serie de 7305
Série 7305
7305 Serie

DeVilbiss[®] Suction Unit Instruction Guide 7305 Series

CAUTION— Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.
Made in U.S.A.

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TABLE OF CONTENTS

Important Safeguards	3
International Travel.....	4
Introduction	5
Important Parts of Your 7305 Series De'Vilbiss® Suction Unit	6
Set-Up	6
Accessory/Replacement Items	7
How to Operate Your 7305 Series De'Vilbiss® Suction Unit.....	10
Battery Charging	12
Cleaning Instructions.....	13
Maintenance.....	14
Filter Changing.....	14
Troubleshooting	14
Specifications/Classifications	16
Warranty	18
Declaration of Conformity.....	19

IEC SYMBOLS

 Attention, consult instruction guide

 Alternating current

 Direct current

 Type BF applied part

 "On" compressor

 "Off" compressor (external battery charging)

 Center positive polarity indicator

IPX2 vertically falling drops shall have no harmful effects when the enclosure is tilted at an angle up to 15° on either side of the vertical.

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IMPORTANT SAFEGUARDS

When using electrical products, especially when children are present, basic safety precautions should always be followed. Read all instructions before using. Important information is highlighted by these terms:

DANGER– Urgent safety information for hazards that will cause serious injury or death.

WARNING– Important safety information for hazards that might cause serious injury.

CAUTION– Information for preventing damage to the product.

NOTE– Information to which you should pay special attention.

READ ALL INSTRUCTIONS BEFORE USING.

SAVE THESE INSTRUCTIONS

▲ DANGER–To reduce the risk of electrocution:

1. Do not use while bathing.
2. Do not place or store product where it can fall or be pulled into a tub or sink.
3. Do not place in or drop into water or other liquid.
4. Do not reach for a product that has fallen into water. Unplug immediately.

▲ WARNING– To reduce the risk of burns, electrocution, fire or injury to persons:

1. Close supervision is necessary when this product is used by, on, or near children or invalids.
2. Use this product only for its intended use as described in this guide.
3. Never operate this product if
 - a. It has a damaged power cord or plug.
 - b. It is not working properly.
 - c. It has been dropped or damaged.
 - d. It has been dropped into water.
 Return the product to an authorized Sunrise Medical service center for examination and repair.

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Important Safeguards (continued)

- Keep the power cord away from heated surfaces.
- Never use while drowsy or asleep.

NOTE– The model 7305D series is not factory equipped with an internal rechargeable battery; it may be purchased separately and installed by your Sunrise Medical provider. 7305P series is factory equipped with an internal rechargeable battery and all information regarding battery operation in this guide is applicable.

⚠ DANGER– The DeVilbiss Suction Unit is a vacuum suction device designed for the collection of nonflammable fluid materials in medical applications only. Improper use during medical applications can cause injury or death. For all medical applications:

- All suctioning should be done in strict accordance with appropriate procedures that have been established by a licensed medical authority.
- Some attachments or accessories may not fit the tubing supplied. All attachments or accessories should be checked prior to use to assure proper fit.

INTERNATIONAL TRAVEL

The 7305P series is equipped with a switch mode power supply allowing operation on any AC voltage (100-240 VAC, 50/60 Hz). However the correct power cord must be used to connect to adaptable wall power.

NOTE– Check power cord for adaptability before using.

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INTRODUCTION

Your DeVilbiss Suction Unit is a compact medical suctioning device which has been designed for reliable, portable operation. Because of the small size, light weight and DC operation, the DeVilbiss Suction Unit is ideal for providing suction in the home, in transport with optional DC cord, or if your model has an internal rechargeable battery, the unit can be operated anywhere. Two collection bottle options give the choice between the standard 800 ml disposable canister or an optional long-term reusable 1,200 ml canister. Following the recommended operating and maintenance procedures outlined in this instruction guide will maximize the life of this product.

Refer to diagram on page 6.

Important Parts of Your 7305 Series DeVilbiss Suction Unit	
Description	Factory equipped on models:
Vacuum regulator knob	All models
Power Switch	All models
DC power input (on side)	All models
Disposable collection bottle	All models
Float shut-off (beneath lid)	All models
Vacuum gauge	All models
Switch mode adapter	Reference matrix page 6
Internal rechargeable battery	7305P series
Tubing, 6'	All models
Carrying case	7305P series
Bacteria filter	All models
Tubing to filter	All models
Connection elbow	All models

NOTE– This unit complies with electromagnetic compatibility standards as defined in the attached declaration of conformity.

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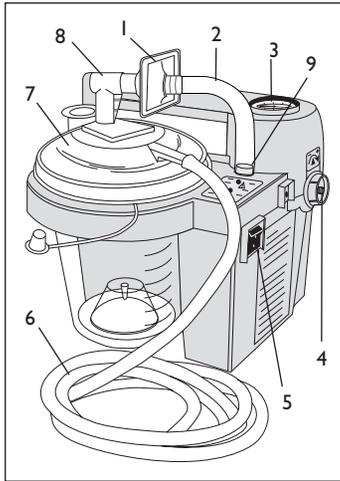
**IMPORTANT PARTS**

1. Bacteria Filter
2. 4" Connection Tubing
3. Vacuum Gauge
4. Vacuum Adjustment
5. Power Switch
6. 6' Patient Tubing
7. 800cc Lid with Canister
8. 90° Elbow Connection
9. Tubing Connector

SET-UP

1. Connect either end of the 4" tubing to the tubing connector then connect the other end to the bacteria filter.
2. The bacteria filter should then be connected to the 90° elbow connection, and the 90° connection should be connected to the top of the canister lid where it says <Vacuum>.
3. The 6' patient tubing should be connected to the canister lid at the outlet labeled <Patient>.
4. Please assure that all connections are secure and without leaks before using.

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ACCESSORY/REPLACEMENT ITEMS

The following items can be purchased separately as accessories or replacement items for your 7305 Series DeVilbiss Suction Unit:

Sunrise Medical Part No.	Description
6305D-611	6' Patient Tubing
7304D-619	12V DC power cord (1each)
7305D-601	1200 ml Reusable collection bottle (20 each)
7305D-602	800 ml Disposable collection bottle (48 each)
7305P-613	Switch mode AC to DC adapter/charger
7305P-614	12V DC high capacity rechargeable battery (1 each)
7305D-606	Carrying case
7305D-607	Replacement gauge
7305D-608	Bacteria filter (12 pack)
7305P-610	Circuit board (7305P series & converted battery units)
7305D-610	Circuit board (7305D series)
7305P-620	High capacity battery conversion kit (battery, 7305P-610 circuit board & battery tray)
7304D-636	Vacuum gauge test fixture
099HD-614	Hospital grade line cord (120 VAC)
7305P-630	Line cord for UK
7305P-631	Line cord for Continental Europe
7305P-632	Line cord for Australia
7305D-603	Filter, 800ml collection bottle, elbow & tubing package
7305D-604	1200ml bottle package

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HOW TO OPERATE YOUR 7305 SERIES DEVILBISS SUCTION UNIT

Before connecting the unit to the AC adapter or accessory 12V DC cord, make sure that the power switch located on the side of the unit is in the "Off" position. Select power source desired.

LED Explanations:

L1 - Green— External power supplied to unit from AC power source or DC cord. Illuminated when external power is supplied.

L2 - Yellow— Battery is being charged. Light will go out when battery is fully charged.

L3 - Red— Low battery. Seek another power source and charge battery as soon as possible when light remains on continuous.

AC OPERATION— Plug the small connector of the AC adapter into the DC power input located on the side of the unit. Plug the AC end into a grounded wall-outlet power source (verify AC voltage listed on AC adapter complies with source).

12V DC OPERATION— (such as a car lighter receptacle). Plug the small connector of the DC power cord into the DC power input on the side of the unit. Plug the large connector into the 12V DC power receptacle of the automobile.

BATTERY OPERATION— Verify that your unit has an internal rechargeable battery; factory installed on models 7305P series or provider installed on models 7305D series (installed as an option). To ensure proper operation from internal battery, fully charge the battery for 10-17 hours as explained in the Battery Charging section. To operate the unit from an internal rechargeable battery, ensure that no external power sources are plugged into the DC power input on the side of the unit.

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Operation... (continued)

Once the power source is selected, simply turn the unit "On" using the power switch located on the side of the unit. The power indicator light (displayed on the top of the unit) will indicate which power source is being used by staying continuously lit when external power is connected.

⚠ WARNING– If the unit does not receive an external power source or the battery does not get recharged immediately, the low battery indicator light will remain on and the performance of the unit will drop off rapidly. Switch to another power source immediately to avoid an interrupted suction procedure.

Adjust the vacuum level from 80 to 550 mm Hg by turning the vacuum regulator knob located on the side of the unit (clockwise to increase vacuum and counter-clockwise to decrease vacuum). The desired level of vacuum can be set by using the gauge located on the top of the unit near the handle. To accurately read the gauge, block the patient end of the hose or cap off the collection bottle and allow the gauge to reach a stable vacuum reading.

NOTE– Gauge is for reference only. If the unit sustains a severe drop, validity of the gauge must be checked.

Suction at the patient is automatically obstructed when liquid level reaches the float shut-off located on the underside of the collection bottle lid.

NOTE– Always transport unit with vacuum adjust knob rotated fully clockwise in case unit is dropped.

CAUTION– When automatic float shut-off is activated, contents of the collection bottle should be emptied. Further suctioning could cause damage to the vacuum pump.

CAUTION– Should fluid be aspirated back into the unit, equipment provider servicing is necessary as possible vacuum pump damage may result.

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BATTERY CHARGING

On 7305P series, the units are equipped with a factory-installed rechargeable battery. The unit will have a light for low battery and charge indication. On 7305D series, the units are not factory equipped with a rechargeable battery; check with your equipment provider to determine if your unit has been upgraded with a rechargeable battery.

Once you've determined that your unit is equipped with an internal rechargeable battery, connect the unit to the proper AC power source using the AC adapter. The green external power light shall be illuminated. The yellow charge indicator will be illuminated while the battery is charging. Verify that this illuminates when charging begins. As the charge nears a full charge, the yellow LED may flash on and off for several minutes. This is normal.

NOTE– A discharged battery will require 10-17 hours (depending on depth of discharge) of charging to reach a full capacity.

NOTE– Do not connect the AC power source to an outlet controlled by a switch to ensure power is supplied to unit at all times.

NOTE– A fully charged battery, on the 7305P series will provide approximately 60 minutes of continuous operation at a zero vacuum level (free flow).

NOTE– If unit is not in use for extended periods, battery should be recharged every 6 months minimum.

CAUTION– Discharging the battery completely will shorten the life of the battery. Do not operate the unit more than a few minutes if the low battery indicator light is lit. Recharge as soon as possible.

NOTE– When charging the battery, the battery charging indicator light will remain lit. If your unit does not hold a charge, please be sure the model you are using has a battery installed prior to returning it to your authorized Sunrise Medical provider or to Sunrise Medical. Verify that the charge light turns on when the external power is applied with the switch in the "Off" position.

NOTE– Units equipped with an internal rechargeable battery contain a sealed lead-acid battery which must be recycled.

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CLEANING INSTRUCTIONS**Collection Bottle:**

1. Shut off unit using power switch and allow vacuum to drop. Disconnect power source from the DC input receptacle on the unit.
2. With the collection bottle still in the holder, remove the lid. The bottle can now be taken out of the holder to be emptied.

NOTE– Collection bottle should be emptied after each use.

3. Collection bottle should be thoroughly cleaned after each use by one of the following three methods:

All Models:

Wash in a hot water/dishwashing detergent solution and rinse with clean, hot tap water. Then wash in one part vinegar to three parts hot water solution. Rinse with hot tap water and air dry.

Permanent Bottle only:

Wash with rubbing alcohol and air dry.

Permanent Bottle only:

Wash with a commercial (bacterial-germicidal) disinfectant, follow disinfectant manufacturer's recommended instructions and dilution rates carefully.

Unit and Collection Lid:

1. With the power switch in the "Off" position, disconnect the DeVilbiss Suction Unit from all external power sources.
2. Wipe the housing and lid with a clean cloth and any commercial (bacterial-germicidal) disinfectant.

CAUTION– Do not submerge in water as this will result in damage to the vacuum pump.

Tubing:

1. Disconnect the tubing from the unit.
2. Tubing should be rinsed thoroughly after every use by running hot tap water through it followed by a solution of one part vinegar to three parts hot water.
3. Rinse with hot tap water and air dry.
4. Keep the outer surface of the tubing clean by wiping with a clean, damp cloth.

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MAINTENANCE

Inspect suction tubing and collection bottle for leaks, cracks, etc. before each use.

⚠ DANGER - Electric shock hazard. Do not attempt to open or remove cabinet, there are no user-serviceable internal components. If service is required, return unit to a qualified Sunrise Medical provider or an authorized service center. Opening or tampering with the unit will void warranty.

Filter Changing:

1. Change bacteria filter if overflow occurs or every two months, whichever comes first.
2. Remove filter by disconnecting it from suction unit and lid assembly.
3. Replace with a clean DeVilbiss bacteria filter (7305D-608 12/pack) and remount to suction unit and lid. Additional filters may be purchased from your authorized Sunrise Medical provider.

NOTE- Do not substitute any other material for this bacteria filter. Substitution may lead to contamination or poor performance; use only DeVilbiss® filters.

TROUBLESHOOTING

NOTE- Your DeVilbiss Suction Unit contains no user-serviceable parts. If you believe your unit is not working properly, BEFORE YOU RETURN IT TO THE HOME MEDICAL EQUIPMENT PROVIDER WHERE YOU PURCHASED IT OR TO Sunrise Medical, please take a few moments to check for these possible causes:

Problem	Action
Unit does not turn on. (Green external power indicator should be illuminated when power is applied.)	<ol style="list-style-type: none"> 1. Check power sources and connections. 2. Ensure wall outlet is live by plugging in a lamp. 3. If running from an internal battery, ensure that your unit has a battery installed. 4. If battery is installed, check that it is fully charged.
Pump runs, but no vacuum.	<ol style="list-style-type: none"> 1. Check that all tubing is connected properly. 2. Check tubing connections for breaks or leaks. 3. Ensure that flow shut-off is not activated. 4. Check for leaks or cracks in bottle assembly.
Low vacuum.	<ol style="list-style-type: none"> 1. Use vacuum adjustment knob to increase vacuum level. 2. Check system for leaks. 3. Push vacuum adjustment knob and then release.
Battery will not hold a charge. (Charge indicator should be illuminated if battery is connected during charge mode.)	<ol style="list-style-type: none"> 1. Ensure that unit is equipped with an internal battery by contacting you Sunrise Medical equipment provider. 2. Verify that charge light turns on. 3. Check electrical connections during charging. 4. Ensure wall outlet is live by plugging in a lamp.

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Weight.....lb. (kg)
7305D Series.....3.8 (1.7)
7305P Series.....6.3 (2.9)

Electrical Requirements - See 7305 Model Matrix on page 6

Internal Rechargeable Battery

7305P Series.....Factory Equipped
7305D Series.....Not Factory Equipped (provider-installed option)

Vacuum Range

7305P & 7305D Series.....80 to 550 mm Hg
Air Flow @ pump inlet: 27 LPM (free flow) typical (may be less when running from internal battery)

Collection Bottle Capacity

7305 D & P Series800 ml (cc) Disposable
7305 D & P Series - Optional1,200 ml (cc) Reusable
Operating Temperature Range.....32°F (0°C) - 104°F (40°C)
Operating Relative Humidity0-95%
Operating Atmospheric Pressure10.2 Psi (70 kPa) - 15.4 Psi (106 kPa)
Storage & Transport Temperature Range, 7305D & P Series-40°F (-40°C) - 158°F (70°C)
Storage & Transport Relative Humidity0-95%
Storage & Transport Atmospheric Pressure7.3 Psi (50 kPa) - 15.4 Psi (106 kPa)

Warranty

7305P Series.....Two-years limited, excluding internal battery and collection bottle
7305D Series.....Two-years limited, excluding collection bottle
Internal Battery.....90-day

Approvals

7305D, & 7305P Series

CAN/CSA-C22.2
No. 601.1-M90
UL 2601-1, CE
EN 606 01-1-2

7305P meets RTCA/D-160D

DO-160D - section 20 Category U
DO-160D - section 21 Category L
For battery operation only
Airline use

Equipment Classifications

With respect to protection from electric shock.....Class I and internally powered
Degree of protection against electric shock.....Type BF Applied Parts
Degree of protection against ingress of liquidsIPX2 and ordinary power supply
Mode of OperationIntermittent Operation: 30 minutes on, 30 minutes off
Equipment not suitable for use in the presence of a flammable anaesthetic mixture with air or nitrous oxide.

ISO Classification

7305P Series only - Electrically powered medical suction equipment for field and transport use according to ISO 10079-1 : 1991
High Flow/High Vacuum
7305D Series - Electrically powered medical suction equipment for non-transport use according to ISO 10079-1 : 1991

Sunrise Medical will make available on request circuit diagrams, parts lists, etc.

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TWO-YEAR LIMITED WARRANTY

The compressor portion of the DeVilbiss® Suction Unit 7305P & 7305D Series (excluding internal rechargeable batteries) is warranted to be free from defective workmanship and materials for a period of two years from date of purchase. Internal rechargeable batteries are warranted for 90 days. Any defective part(s) will be repaired or replaced at Sunrise Medical's option if the unit has not been tampered with or used improperly during that period. Make certain that any malfunction is not due to inadequate cleaning or failure to follow the instructions. If repair is necessary, contact your Sunrise Medical Provider or Sunrise Medical Service Department for instructions: **U.S.A. 800-333-4000**, Canada 905-660-2459, Europe 44-138-444-6688.

NOTE– This warranty does not cover providing a loaner unit, compensating for costs incurred in rental while said unit is under repair, or costs for labor incurred in repairing or replacing defective part(s).

THERE IS NO OTHER EXPRESS WARRANTY. IMPLIED WARRANTIES, INCLUDING THOSE OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED TO THE DURATION OF THE EXPRESS LIMITED WARRANTY AND TO THE EXTENT PERMITTED BY LAW ANY AND ALL IMPLIED WARRANTIES ARE EXCLUDED. THIS IS THE EXCLUSIVE REMEDY AND LIABILITY FOR CONSEQUENTIAL AND INCIDENTAL DAMAGES UNDER ANY AND ALL WARRANTIES ARE EXCLUDED TO THE EXTENT EXCLUSION IS PERMITTED BY LAW. SOME STATES DO NOT ALLOW LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS, OR THE LIMITATION OR EXCLUSION OF CONSEQUENTIAL OR INCIDENTAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY TO YOU.

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

Manufacturer's Note

Thank you for choosing a DeVilbiss® Suction Unit. We want you to be a satisfied customer. If you have any questions or comments, please send them to our address on the back cover.

For Service Call Your Authorized Sunrise Medical Provider:

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Seriennummer _____

DECLARATION OF CONFORMITY

Manufacturer: Sunrise Medical
Address: Respiratory Products Division
 100 DeVilbiss Drive
 Somerset, PA 15501 USA

Product Designation: DeVilbiss® Suction Unit
Type Model: 7305P & 7305D Series

We herewith declare that the above-mentioned product complies with the requirements of EC Directive 93/42/EEC and the following:

Quality System

Standards Applied: ISO9001/EN46001

Notified Body RWTÜV

MDD Annex II Applied

Safety Standards Applied: UL 2601-1-2
 EN 60601.2
 CAN/CSA 22.2
 ISO 10079-1

EMC (Electromagnetic Compatibility)

Compliance to: IEC 801-2 through 5
 CISPR 11/Level B
 EN 60601-1-2

For Battery

Operation Only: DO-160D, Section 20 Category U;
 Section 21 Category L.
 For Airline Use

Place: Somerset, PA

Signature: 
Name:

Michael Chellson

Title: Director of Regulatory and Quality Assurance

European

Contact: Sunrise Medical Ltd.
 Sunrise Business Park
 High Street
 Wollaston, West Midlands DY8 4PS
 ENGLAND
 44-138-444-6688

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Fabricante: Sunrise Medical
Dirección: Respiratory Products Division
 100 DeVilbiss Drive
 Somerset, PA 15501 EE.UU.

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Kontakt in Europa: Sunrise Medical Ltd.
 Sunrise Business Park
 High Street
 Wollaston, West Midlands
 DY8 4PS
 ENGLAND
 44-138-444-6688

Sunrise Medical
Respiratory Products Division
100 DeVilbiss Drive
Somerset, Pennsylvania 15501
USA
814-443-4881

Sunrise Medical Canada, Inc.
237 Romina Drive, Unit 3
Concord, Ontario L4K 4V3
CANADA
905-660-2459

Sunrise Medical Ltd.
Sunrise Business Park
High Street
Wollaston, West Midlands
DY8 4PS
ENGLAND
44-138-444-6688

Sunrise Medical Pty. Limited
15 Carrington Road, Unit 7
Castle Hill NSW 2154
AUSTRALIA
61-2-9899-3144

Sunrise Medical
Division Respiratoire
Zone Industrielle
Route de Meslay
37210 Parçay-Meslay
FRANCE
33-2-47-88-58-36

Sunrise Medical
DeVilbiss Produkte
Industriegebiet
69254 Malsch/Heidelberg
GERMANY
49-7253-980-460